

# ALLOMATRIX<sup>®</sup> CUSTOM

Bone Putty with  
Large Cancellous Chips

Addresses Individual Patient  
Bone Grafting Needs in a Way  
No Pre-Mixed Putty Can

 **WRIGHT**<sup>™</sup>  
FOCUSED EXCELLENCE





MIXABILITY FOR MAXIMUM  
**Flexibility**



**Bone Marrow Aspirate.**  
*Add Osteogenic Potential. (Optional)*



**Control Cancellous Content.**  
*Dial-in Desired Structural Consistency.*

# ALLOMATRIX® CUSTOM


## Bone Putty with Large Cancellous Chips

### RAW FLEXIBILITY TO MEET PATIENT PATHOLOGY NEEDS

Surgeons desire the independence and flexibility to custom blend graft materialsto control structural consistency. ALLOMATRIX® CUSTOM Bone Graft Putty delivers. With features that include an osteoinductive Demineralized Bone Material (DBM), a osteoconductive carrier medium, and the ability to reconstitute in the OR, ALLOMATRIX® CUSTOM Bone Graft Putty meets individual patient bone grafting needs in a way no pre-mixed putty can.

### OPTIMUM CANCELLOUS CHIP CONTENT — YOUR CHOICE

ALLOMATRIX® CUSTOM Bone Graft Putty incorporates DBM and the highest cancellous chip content on the market – both recovered from the same donor, which is blended at the time of surgery. The result is a custom product that can range from minimum to maximum CBM content. ALLOMATRIX® CUSTOM Putty provides flexibility to augment the graft with optional bone marrow aspirate without compromising handling characteristics.

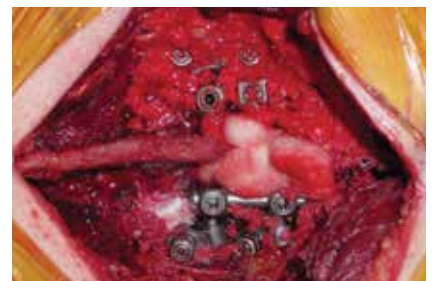

*Actual ALLOMATRIX® CUSTOM product mixed by surgeon and placed in situ.*






**Bone Marrow Carrier Medium**

**MARROW VOLUME | Recommendations**

| ALLOMATRIX® CUSTOM Volume (dry powder) | Bone Marrow Volume |
|--|--------------------|
| 5 cc                                   | 3 cc               |
| 10 cc                                  | 6 cc               |

**NOTE |** These approximations were obtained using anticoagulated fluids. It is suggested that the user begin with the amounts in the table, and add additional fluids to achieve desired handling characteristics. Additional information is included with the labeling accompanying the product.

MINIMUM TO MAXIMUM  
**Structural Consistency**

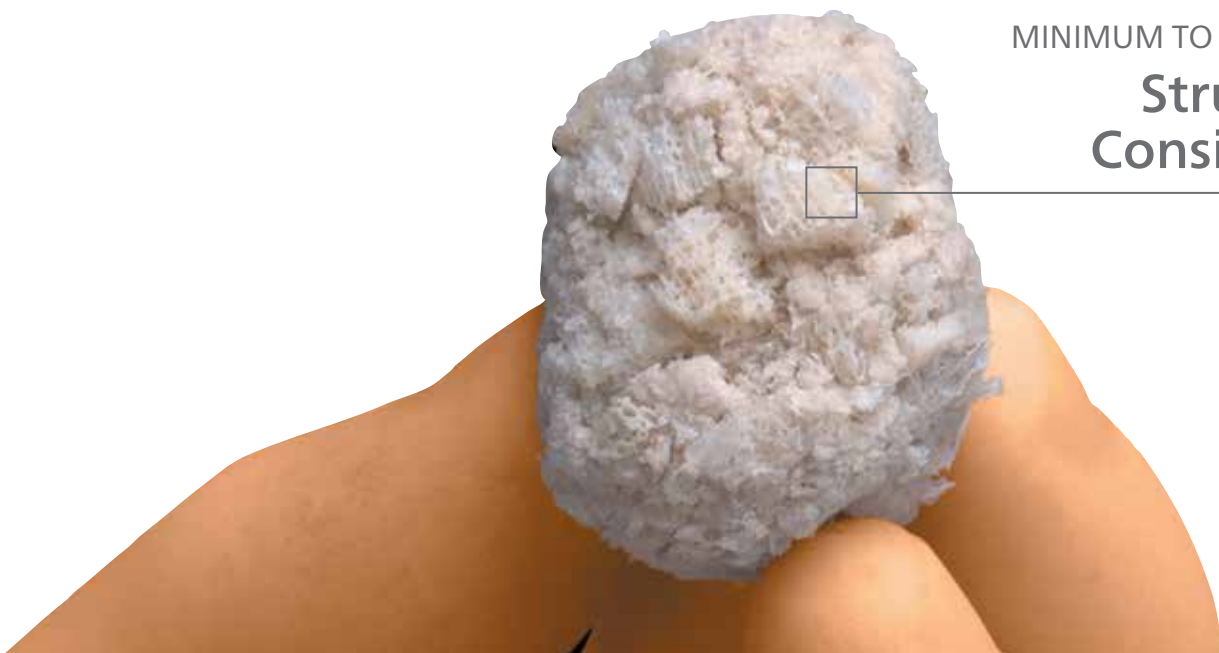

## ORDERING INFORMATION

### OSTEOGENIC SOLUTIONS

#### ALLOMATRIX® CUSTOM Bone Putty with Large Cancellous Chips



|   |       |      |
|---|-------|------|
| 86XC-0500<br>ALLOMATRIX® CUSTOM<br>Bone Putty | 5 cc  | 1 ea |
| 86XC-1000<br>ALLOMATRIX® CUSTOM<br>Bone Putty | 10 cc | 1 ea |



Wright Medical Technology, Inc.  
1023 Cherry Road  
Memphis, TN 38117  
800 238 7117  
901 867 9971  
www.wmt.com

Wright Medical EMEA  
Atlas Arena, Australia Building  
Hoogoorddreef 7  
1101 BA Amsterdam  
the Netherlands  
011 31 20 565 9060

\*Dry ingredients are pre-mixed. Putty appearance varies with mixing.  
™Trademarks and ®Registered marks of Wright Medical Technology, Inc.  
©2013 Wright Medical Technology, Inc. All Rights Reserved.

008892A 09-Nov-2013